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GOeasy Ca-Line LC

Light cure radiopaque calcium hydroxide liner

GOeasy Ca-Line LC is a light cure, radiopaque calcium hydroxide liner.

Indications

- indirect pulp capping
- lining of cavities for following filling procedures

Contraindications

Do not use GOeasy Ca-Line LC if the recommended working technique is not possible or if the patient is known to be allergic to any of the ingredients.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let it allow coming into contact with materials containing eugenol.

Application

Clean and dry the cavity thoroughly. Avoid contamination.

Screw off the cap of the syringe and substitute it by an application needle.

Note

In direct proximity to the pulp (direct pulp capping) application of an aqueous calcium hydroxide liner (e.g. GOeasy Ca-Line) is recommended.

To enhance bonding of GOeasy Ca-Line LC use a bonding agent. Etching with GOeasy Etch and application of GOeasy Basic Bond according to the corresponding instructions is recommended. In areas with a thin dentin layer over the pulp do without bonding (pulp protection).

Apply GOeasy Ca-Line LC in a thin layer and light cure for 40 sec. In case of layers more than 1 mm cure each layer separately for 40 sec.

After curing apply the filling according to manufacturer instructions.

Storage

Store at 4 - 25 °C (39 - 77 °F) protected from light and moisture. Close syringe immediately after use.

Additional notes

- When using GOeasy Ca-Line LC wear protective glasses and cover the patient eyes.
- Unpolymerized materials may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- In case of contact with eyes, rinse with copious amounts of water and consult a physician immediately.
- Avoid contact with skin. If accidental contact occurs, wash immediately with water.
- Do not use any resin to adjust viscosity of GOeasy Ca-Line LC.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental professional use only!

The instructions for use has to be kept for the duration of the application.

Caution

Federal law restricts this device to sale by or on the order of a dentist.