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GOeasy Ca-Line

Calcium hydroxide liner

GOeasy Ca-Line is a radiopaque water-based calcium hydroxide liner.

Indications

- Indirect capping for the treatment of caries profunda.
- Direct capping when the pulp is opened or a pulpotomy is performed.
- Lining of cavities to prevent exposure to acid media when using cements.
- Temporary filling of root canals if root growth has not yet been terminated.

Preparation of GOeasy Ca-Line syringe

Screw off the cap and substitute it by the application needle. **Store the screw cap!** If desired, the application needle might be bend in a slight arc to facilitate the application.

Handling precaution

Squeeze out the first small drop on a pad to ensure proper handling and prevent „shooting“ out of the GOeasy Ca-Line by entrapped air. This must be done for each new application.

Application

Clean and dry the cavity.

Apply GOeasy Ca-Line in thin layers until the desired thickness is reached. Allow each layer to dry briefly or dry carefully with an oil and water free stream of air. Do not apply to the cavity edges and remove any paste sticking to them. Then proceed with adhesive, underfilling and filling procedures.

For root canal fillings, first prepare and clean the root canal (hydrogen peroxide 3% or sodium hypochlorite solution) and insert GO easy Ca-Line paste with a suitable instrument. Avoid overfilling (apical) and trapping of air. It is advisable to inspect the root canal filling by taking an X-ray.

Important notes

- The material hardens when exposed to air.
- Immediately after use replace the cap.
- For each application take a **new application needle**.
- **Do not use hardened paste.** If hardening still occurs inside the application needle, use a new one.

Side-effects

If the root canal is overfilled, discomfort may result (sensitivity for pressure and/or swelling) but, as a rule, this will subside within 24-48 hours.

Contra-indications or interactions are not known.

Warnings

- When using GOeasy Ca-Line wear protective glasses and cover the patient eyes.
- GOeasy Ca-Line is irritant. In case of contact with eyes a risk of serious damage to eyes exists.
- Avoid contact with skin, mucous membrane and eyes.
- In case of contact with eyes, rinse with copious amounts of water and consult a physician immediately.
- If accidental skin contact occurs, wash immediately with water.

Storage

Do not store at less than +8° C!

Do not use after expiry date.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture.

P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law,

P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental professional use only!

The instructions for use has to be kept for the duration of the application.

Caution:

Federal law restricts this device to sale by or on the order of a dentist.