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Instructions for Use EN

GOeasy Etchbond Mono

One-component self-etching light cure adhesive

GOeasy Etchbond Mono is a simple to use self-etching light curing one-component adhesive. It is designed for strong bonding of composites to enamel and dentin.

GOeasy Etchbond Mono is compatible with all current brands of visible light cure composite restorative materials.

Contraindications

Do not use GOeasy Etchbond Mono if the recommended working technique is not possible or if the patient is known to be allergic to any of the ingredients.

Side effects

In singular cases, GOeasy Etchbond Mono may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used. Irritations resulting from direct contact with the pulp cannot be ruled out.

Therefore for pulp protection the cavity floor in cases of deep excavations should be covered with a thin layer of calcium hydroxide material.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of GOeasy Etchbond Mono. Neither store the material in proximity of eugenol containing products, nor let the material allow coming into contact with materials containing eugenol.

Application

1. Isolation

Rubber dam is the recommended method of isolation.

2. Cavity preparation

Clean the tooth with flour of pumice and water prior to preparation. Prepare the cavity with minimal tooth reduction. Margins should have a slight **(0.5 - 1.0 mm)** bevel placed in the enamel to increase the surface area for greater bond strength.

3. Pulp protection

Cavity floor of deep excavations should be covered with a thin layer of calcium hydroxide material (e.g. GOeasy Ca-Line).

4. Application of

GOeasy Etchbond Mono

Apply GOeasy Etchbond Mono with a brush onto the moist enamel and dentin surfaces for 30 seconds with agitation.

The material should build a homogeneous layer. Air thin gently for 10 seconds to remove the volatile components and to disperse the adhesive. Then light cure with a suitable dental light unit for 20 seconds before placement of a composite.

5. Restorative placement

Apply the restorative material according to the instructions of the manufacturer. Best results are obtained with application of a thin layer of a flowable composite (e.g. GOeasy Flow) followed by the application of a moldable composite (e.g. GOeasy Fill). Light cure each composite layer separately according to the corresponding user instructions.

Warnings

- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates
- Avoid contact with skin, mucous membrane and eyes
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.

Storage

Do not store above 25 °C (77 °F)!
Do not use after expiry date.
Avoid storage in direct sunlight.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any

implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental professional use only!

The instructions for use has to be kept for the duration of the application.

Caution:

Federal law restricts this device to sale by or on the order of a dentist.