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### GOeasy F-Mix Gel

#### Fluoride containing gel

**GOeasy F-Mix Gel** is a fluoride containing gel with Xylitol. For desensitizing of teeth it is applied on dentine and enamel surface. GOeasy F-Mix Gel contains in addition to easy soluble fluorides Calciumfluoride and Fluorapatite. Fluorapatite particles are known to enhance remineralization of tooth material. GOeasy F-Mix Gel has a pleasant sweetish taste.

GOeasy F-Mix Gel adheres to wet and dry surfaces and can be applied easily and time saving.

1 ml GOeasy F-Mix Gel contains 22600 ppm fluoride.

#### GOeasy F-Mix Gel is designed for the following indications:

- protection and fluoridation of tooth surface
- treatment of hypersensitive teeth
- treatment of exposed dental necks
- sealing of teeth during and/or after orthodontic procedures
- sealing of tooth surface after bleaching
- GOeasy F-Mix Gel is for adults and for children over 3 years old

#### Application of GOeasy F-Mix Gel

##### Recommended dosage:

For treatment with GOeasy F-Mix Gel the following dosage is recommended for a complete dentition:

**Teeth of 1. dentition:** approx. 0.2 ml GOeasy F-Mix Gel

**Permanent teeth:** approx. 0.4 ml GOeasy F-Mix Gel

From experience, the required amount for a complete set of teeth ranges between 0.2 and 0.4 ml GOeasy F-Mix Gel.

A strand length of 1 cm on the mixing pad is equivalent to approx. 0.1 ml GOeasy F-Mix Gel.

**For children from 3 to 6 years ensure that not more than 0.2 ml GOeasy F-Mix Gel is applied.**

GOeasy F-Mix Gel should be applied as a thin film on the dental hard tissue. A thick film give no therapeutic advantages but peels off more easily.

Clean the teeth thoroughly from tartar and remove excess moisture and saliva from the areas to be treated. Place GOeasy F-Mix Gel on the mixing pad and apply it with the supplied brush on the teeth. The entire surface should be covered with an even thin film. Moisturizing by natural saliva ensures a proper setting of GOeasy F-Mix Gel.

##### Note:

During the first 4 hours after application of GOeasy F-Mix Gel avoid solids, alcohol, tooth cleaning with brush and floss.

Close tube immediately after use.

##### Remarks

- GOeasy F-Mix Gel contains natural resins.
- In case of known hypersensitivities (allergic reactions) to natural resins, GOeasy F-Mix Gel should not be used.
- Natural resins may inhibit the polymerization and adhesion of composites.
- To prevent the cap to stick, wipe-off any contaminating residues on the screw of the tube with an ethanol-containing cloth.

- GOeasy F-Mix Gel is contraindicated in patients with ulcerative gingivitis or stomatitis. GOeasy F-Mix Gel is not intended for systemic treatments.
- In case of protecting teeth with GOeasy F-Mix Gel during application of orthodontic multi-band technique, the brackets are placed on the teeth before the application of GOeasy F-Mix Gel.
- In case of hypersensitivity reactions, remove GOeasy F-Mix Gel with a soft tooth brush and floss and rinse with warm water.
- On the day of treatment use no other fluoride preparations. Stop treatment with fluoride tablets for several days.
- Do not swallow GOeasy F-Mix Gel.
- In case of swallowing large amounts of GOeasy F-Mix Gel consult a doctor immediately.
- Do not use GOeasy F-Mix Gel for children under 3 years.

##### Storage

Do not store above 25 °C (77 °F)! Do not use after expiration date!

##### Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture.

P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

##### Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

##### Keep away from children!

##### For dental professional use only!

The instructions for use has to be kept for the duration of the application.

##### Caution:

**Federal law restricts this device to sale by or on the order of a dentist.**