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Instructions for Use EN

GOeasy F-Mix Varnish Fluoride containing varnish

GOeasy F-Mix Varnish is a fluoride containing varnish. For desensitizing of teeth it is applied on dentine and enamel surface. GOeasy F-Mix Varnish contains in addition to easy soluble fluorides medium soluble Calciumfluoride and low soluble Fluorapatite. Fluorapatite particles are known to enhance remineralization of tooth material. GOeasy F-Mix Varnish adheres to wet and dry surfaces and can be applied easily and time saving. 1 ml GOeasy F-Mix Varnish contains 22600 ppm fluoride.

GOeasy F-Mix Varnish is designed for the following indications:

- protection and fluoridation of tooth surface
- treatment of hypersensitive teeth
- treatment of exposed dental necks
- sealing of teeth during and/or after orthodontic procedures
- sealing of tooth surface after bleaching
- GOeasy F-Mix Varnish is indicated for use in adults and in children (over 3 years old)

Application of GOeasy F-Mix Varnish

Before application vigorous shake the bottle with the GOeasy F-Mix Varnish. During shaking listen to the glass bead inside the bottle. After shaking, the suspension should be really thin and flow easily from the dropper.

Recommended dosage:

For treatment with GOeasy F-Mix Varnish the following dosage is recommended for a complete dentition:

Teeth of 1. dentition: approx. 0.2 ml (approx. 10 drops) GOeasy F-Mix Varnish

Permanent teeth: approx. 0.4 ml (approx. 20 drops) GOeasy F-Mix Varnish

For children from 3 to 6 years ensure that not more than 0.2 ml GOeasy F-Mix Varnish is applied.

Clean the teeth thoroughly from tartar and remove excess moisture and saliva from the areas to be treated.

Place GOeasy F-Mix Varnish in a mixing well and apply it with the supplied Micro-Applicator on the prepared teeth. The entire surface should be covered with an even thin film. GOeasy F-Mix Varnish should be applied as a thin film on the dental hard tissue. A thick film give no therapeutic advantages but peels off more easily. Leave in GOeasy F-Mix Varnish for 10-20 seconds, then dry with air.

Close bottle immediately after use.

Note:

During the first 4 hours after application of GOeasy F-Mix Varnish avoid solids, alcohol, tooth cleaning with brush and floss.

Remarks

- GOeasy F-Mix Varnish contains natural resins.

- In case of known hypersensitivity (allergic reactions) to natural resins, GOeasy F-Mix Varnish should not be used.
- Natural resins may inhibit the polymerization of composites and the adherence of adhesives.
- To prevent the cap to stick, wipe-off any contaminating residues on the screw of the tube with an ethanol-containing pad.
- GOeasy F-Mix Varnish contains ethylacetate. Ethylacetate is highly flammable. Keep away from sources of ignition. After eye contact rinse out with plenty of water und call an ophtalmologist. After contact with skin, wash with plenty of water and soap.
- In patients with ulcerative gingivitis and stomatitis, GOeasy F-Mix Varnish should not be used. GOeasy F-Mix Varnish is not intended for systemic treatments.
- In case of protecting teeth with GOeasy F-Mix Varnish during application of orthodontic multi-band technique, the brackets are placed on the teeth before the application of GOeasy F-Mix Varnish.
- In case of hypersensitivity reactions, remove GOeasy F-Mix Varnish with a soft tooth brush or dental floss. Rinse with warm water.
- On the day of treatment use no other fluoride preparations. Stop treatment with fluoride tablets for several days.
- GOeasy F-Mix Varnish should not be swallowed.
- Do not use GOeasy F-Mix Varnish for children under 3 years.

Storage

Do not store above 25 °C (77 °F)! Do not use after expiration date!

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental professional use only!

The instructions for use has to be kept for the duration of the application.

Caution:

Federal law restricts this device to sale by or on the order of a dentist.