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GOeasy Fill Ceram

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Instructions for Use EN

GOeasy Fill Ceram

Light cure microhybrid composite, for high stress areas

GOeasy Fill Ceram is a visible light cure microhybrid composite for anterior and posterior restorations. It is based on BIS-GMA-resin and inorganic filler particles of 0.05-0.9 µm. Filler content: 81 % (wt), 63 % (vol.). The composite material meets the requirements of ISO 4049.

Indications of GOeasy Fill Ceram

Suitable for all cavities

Contraindications

The placement of GOeasy Fill Ceram is contraindicated

- If a dry working area or the recommended application technique are not possible
- If the patient is known to be allergic to any of the ingredients in GOeasy Fill Ceram.

Side effects

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection the cavity floor in cases of deep excavations should be covered with a thin layer of calcium hydroxide material.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

Preparation of tooth and color matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching.

Ascertain the tooth shade while teeth are still moist and select the appropriate GOeasy Fill Ceram shade.

GOeasy Fill Ceram is shaded according to VITA®-Shades.

Cavity preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction. Deep excavations should be covered with a thin layer of a calcium hydroxide liner (e.g. GOeasy Ca-Line). Areas next to the pulp should be optimally disinfected by photo activation (PAD). Condition and apply a bonding agent according to the manufacturer instructions. The total etch technique with GOeasy Etch and subsequent application of a light cure bonding agent (e.g. GOeasy Basic Bond) is recommended. Otherwise use a self etching bonding agent (e.g. GOeasy Etchbond Mono or GOeasy Etchbond Duo) before application of the GOeasy Fill Ceram.

Application and curing of GOeasy Fill Ceram

Apply GOeasy Fill Ceram in the selected shade. GOeasy Fill Ceram is easy to model. Place it by using a suitable instrument. Transparent matrix strips may be used. In case of deep and extended cavities best results are obtained with a build-up of a base liner using a light cure resin modified cement or a light cure flowable composite (e.g. GOeasy Flow), followed by the application of GOeasy Fill Ceram. In case of small cavities GOeasy Fill Ceram

can be applied directly.

For an optimum result apply GOeasy Fill Ceram in **layers of max. 2 mm**. Light cure each increment separately.

By using a polymerization unit with a **light intensity of at least 1000mW/cm²**, cure each increment as follows:

Lighter shades (e.g. A1, A2, C2) **20 seconds**

Darker shades (e.g. A3.5, B3) **30 seconds**

Finishing

Contour with fine diamonds, stones or burs. Polish to high gloss with discs or rubber points. Interproximal finishing is accomplished by fine grit finishing strips.

Storage

Do not store above 25 °C (77 °F).

Protect from direct sunlight. Do not use after expiry date.

Close syringe immediately after use to avoid exposure to light.

GOeasy Fill Ceram Tips are for single use only.

Additional notes

- Unpolymerized materials may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.

- Do not use any resin to adjust viscosity of composite restorative material.

- VITA® is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture.

P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental professional use only!

The instructions for use has to be kept for the duration of the application.

Caution:

Federal law restricts this device to sale by or on the order of a dentist.