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GOeasy Flow Nano

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GO easy Flow Nano

Light cure flowable nanohybrid composite

GO easy Flow Nano is a visible light cure high-viscous flowable nanohybrid composite for anterior and limited posterior restorations. Beside its low shrink and low abrasion GO easy Flow Nano shows improved mechanical properties. GO easy Flow Nano is based on BIS-GMA-resin and inorganic filler particles <1,0 µm. The composite material meets the requirements of ISO 4049.

Indications of GO easy Flow Nano

- Anterior restorations (class III, IV, V)
- Small class I and II restorations
- Lining of cavities

Contraindications

The placement of GO easy Flow Nano is contraindicated

- If a dry working area or the recommended application technique are not possible
- If the patient is known to be allergic to any of the ingredients in GO easy Flow Nano.

Side effects

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection the cavity floor in cases of deep excavations should be covered with a thin layer of calcium hydroxide material.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the

composite allow coming into contact with materials containing eugenol.

Preparation of tooth and color matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching. Ascertain the tooth shade while teeth are still moist and select the appropriate GO easy Flow Nano shade. GO easy Flow Nano is shaded according to VITA®-Shades.

Cavity preparation

After isolation (best with rubber dam) prepare the cavity with minimal tooth reduction. Deep excavations should be covered with a thin layer of a calcium hydroxide liner (e.g. GO easy Ca-Line). Areas next to the pulp should be optimally disinfected by photo activation (PAD). Condition and apply a bonding agent according to the manufacturer instructions. The total etch technique with GO easy Etch and subsequent application of a light cure bonding agent (e.g. GO easy Basic Bond) is recommended. Otherwise use a self etching bonding agent (e.g. GO easy Etchbond Mono or GO easy Etchbond Duo) before application of the GO easy Flow Nano.

Application and curing of GO easy Flow Nano Apply GO easy Flow Nano in the selected shade. Place it by using a suitable instrument. Transparent matrix strips may be used.

For an optimum result apply GO easy Flow Nano in **layers of max. 1 mm**. Light cure each increment separately. By using a polymerization unit with a light intensity of at least 1000mW/cm², cure each

increment as follows:

Lighter shades (e.g. A1, A2, C2) **20 seconds**

Darker shades (e.g. A3.5, B3) **30 seconds**

Finishing

Contour with fine diamonds, stones or burs. Polish to high gloss with discs or rubber points. Interproximal finishing is accomplished by fine grit finishing strips.

Storage

Do not store above 25 °C (77 °F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

Additional notes

- Unpolymerized materials may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Do not use any resin to adjust viscosity of composite restorative material.
- VITA® is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture.

P.L. Superior Dental Materials GmbH makes

no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental professional use only!

The instructions for use has to be kept for the duration of the application.

Caution:

Federal law restricts this device to sale by or on the order of a dentist.