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GOeasy Provi Fill

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GOeasy Provi Fill

Light cure temporary filling material

GOeasy Provi Fill is a one component light cure temporary filling material. The elasticity after curing makes removal of GOeasy Provi Fill simple and quick, even if small undercuts are present. Damage to preparation margins is thereby avoided.

Indications

- Temporary restorations, in particular for inlays and onlays
- Re-line material for prefabricated temporary crowns and bridges made of polycarbonate and methacrylate

Contraindications

If the patient is known to be allergic to any of the ingredients.

Side effects

In singular cases, GOeasy Provi Fill may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of GOeasy Provi Fill. Neither store the material in proximity of eugenol containing products, nor let the material allow coming into contact with materials containing eugenol.

Application

Preparation of teeth

Preparation, lining and taking an impression

are done as usual.

If desired, a matrix can be applied.

Clean cavity with water spray and dry.

Placement of GOeasy Provi Fill

It is recommended to apply a thin layer of a calcium hydroxide liner (e.g. **GOeasy Ca-Line**) into the cavity before application of GOeasy Provi Fill.

Place a sufficient portion of GOeasy Provi Fill into the cavity using a spatula or another suitable instrument. Contour leaving minimum excess.

If GOeasy Provi Fill sticks to the application instrument, slightly wet the instrument with light cure bonding material (e.g. **GOeasy Basic Bond**).

Curing

Light cure for **40 seconds** with a dental halogen light unit or an LED (wavelength 400–500 nm, light intensity min. 1000 mW/cm²). Layers up to 5 mm cure without problems. Deeper cavities should be filled and light cured in layers.

Remove matrix. Check occlusion.

Removing after use

For removing after use of the temporary filling insert a strong instrument into the GOeasy Provi Fill and remove.

Important notes

- GOeasy Provi Fill generally remains in place in standard inlay preparations. If mechanical retention is minimal, GOeasy Provi Fill can be cemented with an eugenol-free cement. If GOeasy Provi Fill is applied to the undercuts of the proximal region, retention

can be improved. In this case, do not use a matrix, but a wooden interdental wedge before applying GOeasy Provi Fill.

- If light cure base or lining materials have been used, isolate them with vaseline or an other separating medium, because GOeasy Provi Fill can bond to them.
- GOeasy Provi Fill can be adjusted with silicone rubber finishers or round steel burs and polished with silicone rubber polishers. Excess can also be removed with a scalpel.
- To increase stability of the temporary filling permanent light cure composite can be used for the occlusal section. Light cure GOeasy Provi Fill separately or together with the filling material.
- Unsuitable as sole material for crowns and bridges.

Precautions

- Contact of GOeasy Provi Fill with skin and eyes should be avoided.
- In case of contact with skin and eyes rinse immediately with plenty of water and consult a physician if necessary.

Storage

Do not store above 25 °C (77 °F). Cool storage of 2-8 °C (36-46 °F) is indicated if the material is not used regularly. Close syringe immediately after use. Do not use after expiry date.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a

particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental professional use only!

The instructions for use has to be kept for the duration of the application.

Caution:

Federal law restricts this device to sale by or on the order of a dentist.