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### GOeasy Reprint Light – Automix 1:1 Superhydrophiles Korrekturabform- material, additionsvernetzendes Silikon – niedrige Viskosität

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## Instructions for Use EN

### GOeasy Reprint Light – Automix 1:1 Super hydrophilic correction impression material based on addition curing vinyl silicones – low viscosity

GOeasy Reprint Light is a strong hydrophilic correction impression material for the putty/ wash technique in combination with GOeasy Reprint Putty.

GOeasy Reprint Light is a low viscosity precision impression material based on addition curing vinyl silicones delivered in Automix-cartridges. It can be extruded directly onto the teeth or the tray. GOeasy Reprint Light shows strong hydrophilic properties. Therefore it improves the wettability and yields an exact reproduction of subgingival areas and other moist surfaces.

GOeasy Reprint Light is thixotropic. That means it stays in the position in the tray and on the tooth, but under pressure it flows. It meets the requirements of ISO 4823, Type 3 (light-bodied consistency).

### Preparation of the Automix-Cartridge

Mount the cartridge into the application gun. Remove and dispose the closure-cap. Install one of the supplied mixing cannulas onto the cartridge and extrude needed material.

### Note:

Store used cartridge with fixed used mixing cannula. **Do not re-use the cartridge cap.** Closing the cartridge by turning on the cap between use could cause cross contamination of components and thus premature setting.

### Impression taking

Load the auto mixed material onto the tray or directly onto the teeth. Insert the tray not later

than **2:00 min** (working time at 23°C or 74°F) after extruding GOeasy Reprint Light into the mouth. Maintain position for at least **3:00 minutes**.

### Casting

The impression should be stored under normal room conditions (max. 25°C or 78°F). The impression can be poured **30 minutes** after removal from the mouth. No other time limitation does exist.

### Electroplating

All impressions can be copper- or silverplated.

### Disinfection

Impressions can be disinfected with standard disinfection solutions.

### Storage

Do not store above 25 °C (77 °F). Do not use after expiry date.

### Additional notes:

Do not use with standard (condensation reaction type) silicone impression materials.

Do not use Latex or sulfur-containing polymeric gloves, they may interfere with the setting reaction of the impression material.

### Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture.

P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials

GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

### Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

### Keep away from children!

### For dental professional use only!

The instructions for use has to be kept for the duration of the application.

### Caution:

**Federal law restricts this device to sale by or on the order of a dentist.**