



## Instructions for Use EN

### GOeasy Reprint Putty

Kneadable base impression material based on addition curing vinyl silicones

**GOeasy Reprint Putty** is used as a base impression material for the putty/wash impression technique in combination with **GOeasy Reprint Light** or **GOeasy Reprint Medium**.

**GOeasy Reprint Putty** is a kneadable precision impression material based on addition curing vinyl silicone. **GOeasy Reprint Putty** is easy to mix and not sticky. It meets the requirement ISO 4823, type 0 (putty consistency).

### Application

Perforated or non-perforated metal or plastic trays may be used with **GOeasy Reprint Putty**. In order to improve adhesion of **GOeasy Reprint Putty** to the tray, a silicone adhesive according to the corresponding instructions.

Use **equal parts** (volume or weight) of **base paste** and **catalyst paste**. Take the base paste with the dark measuring spoon and the catalyst paste with the white measuring spoon.

Knead until the colour is homogeneous (at least **30 seconds**). Load the ready kneaded material into the prepared tray. Insert the loaded tray in the mouth not later than **1:30 minutes** (working time at 23 °C or 74 °F) after start of mix. Maintain position for at least **3 minutes**.

The impression should be stored at normal room temperature (max. 25 °C or 77 °F).

### Casting

The impression can be poured **30 minutes** after removal from the mouth. No other time limitation does exist.

### Electroplating

All impressions can be copper- or silverplated.

### Desinfection

Impressions can be desinfected with standard desinfection solutions.

### Storage

Do not store above 25 °C (77 °F). Do not use after expiry date

### Additional notes:

Do not interchange lots of pots. Do not interchange measuring spoons.

Do not use with standard (condensation reaction type) silicone impression materials.

Do not use Latex gloves, they may interfere with the setting reaction of the impression material.

### Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

### Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

### Keep away from children! For dental professional use only!

The instructions for use has to be kept for the duration of the application.

**Caution:**  
**Federal law restricts this device to sale by or on the order of a dentist.**

